




This guide provides first time setup instructions for the epoc Reader. For complete user instructions, please refer to the epoc System Manual.

Package Contents

- epoc Reader
- AC Adapter power supply
- Sheet of labels
- Sheet of clear label overlays

epoc Reader Components

The figure below shows the Reader's primary components and status light indicators.

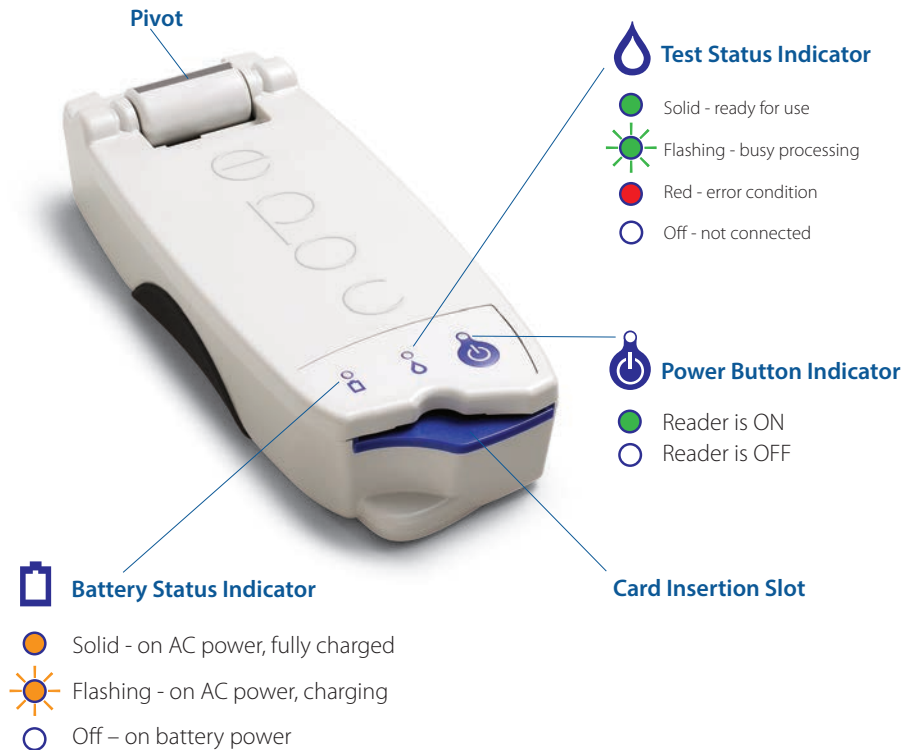

Manufactured by:
Epocal Inc.
2060 Walkley Road
Ottawa, ON K1G 3P5 Canada
1.613.738.6192
1.613.738.6195 Fax

Distributed by:
Alere North America, LLC
30 S. Keller Road, Suite 100
Orlando, FL 32810 USA
1.877.441.7440 (US)
1.321.441.7200 (Outside US)

Authorized Representative:
Emergo Europe
Molenstraat 15, 2513 BH
The Hague, Netherlands
+31.70.345.8570
+31.70.346.7299 Fax




This guide provides first time setup instructions for the epoc Reader. For complete user instructions, please refer to the epoc System Manual.

Package Contents

- epoc Reader
- AC Adapter power supply
- Sheet of labels
- Sheet of clear label overlays

epoc Reader Components

The figure below shows the Reader's primary components and status light indicators.

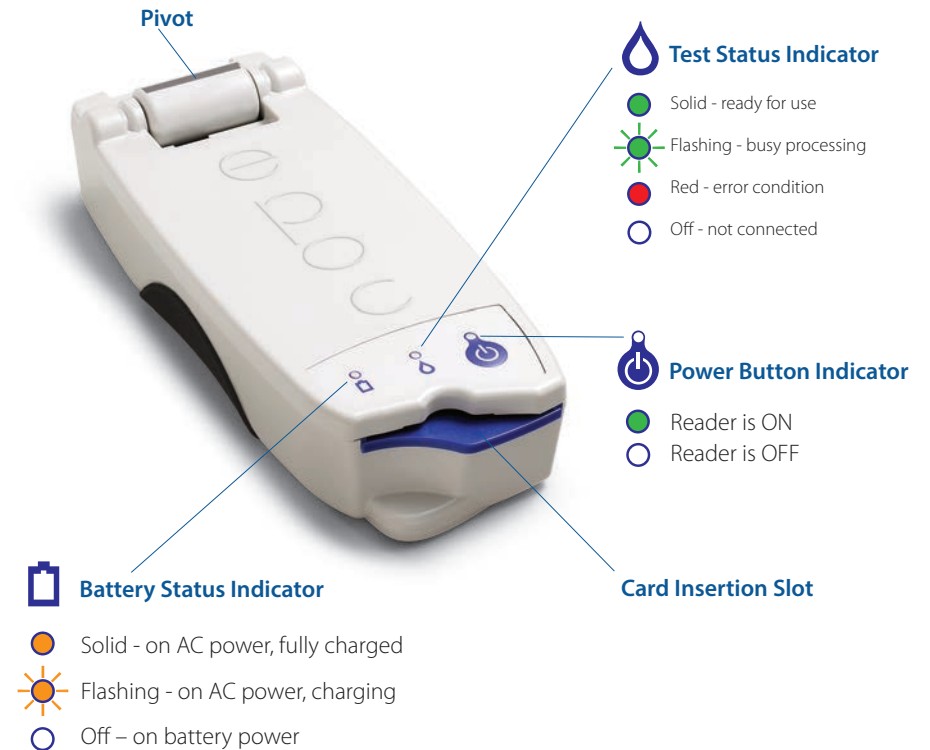

Manufactured by:
Epocal Inc.
2060 Walkley Road
Ottawa, ON K1G 3P5 Canada
1.613.738.6192
1.613.738.6195 Fax

Distributed by:
Alere North America, LLC
30 S. Keller Road, Suite 100
Orlando, FL 32810 USA
1.877.441.7440 (US)
1.321.441.7200 (Outside US)

Authorized Representative:
Emergo Europe
Molenstraat 15, 2513 BH
The Hague, Netherlands
+31.70.345.8570
+31.70.346.7299 Fax




Prior to use . . .

Step 1: Charge

Before using the epoc Reader, charge its battery as follows:

1. Make sure the Reader is OFF. If the Power Button Indicator is green, press and hold the power button for at least three seconds to turn the Reader off.
2. Plug the AC Adapter into a wall socket.
3. Plug the AC Adapter into the power jack on the back of the Reader.
 - The amber battery status indicator will start blinking
 - When the battery has been fully charged, the amber battery status indicator will stop blinking

Note: Initial charge will take approximately four hours.

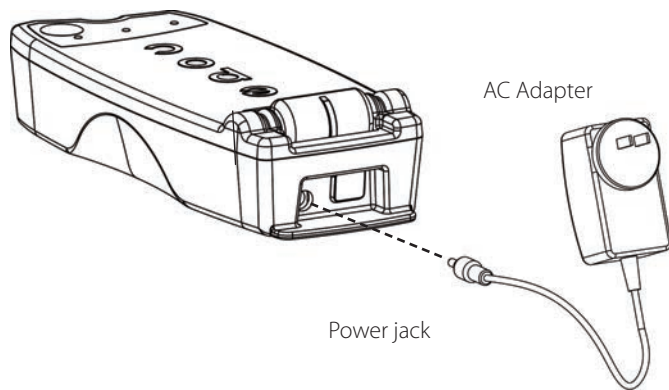

Step 2: Label

Label the epoc Reader to help easily identify it from other Readers in your facility. Use the included labels, and place the clear overlay over the label to increase its durability.

Note: Before placing this unit into service, consult your facility's internal quality control procedures and regulatory requirements.

© 2015 Alere. All rights reserved. The Alere Logo, Alere, and epoc are trademarks of the Alere group of companies. All other trademarks referenced are trademarks of their respective owners.


Prior to use . . .

Step 1: Charge

Before using the epoc Reader, charge its battery as follows:

1. Make sure the Reader is OFF. If the Power Button Indicator is green, press and hold the power button for at least three seconds to turn the Reader off.
2. Plug the AC Adapter into a wall socket.
3. Plug the AC Adapter into the power jack on the back of the Reader.
 - The amber battery status indicator will start blinking
 - When the battery has been fully charged, the amber battery status indicator will stop blinking

Note: Initial charge will take approximately four hours.

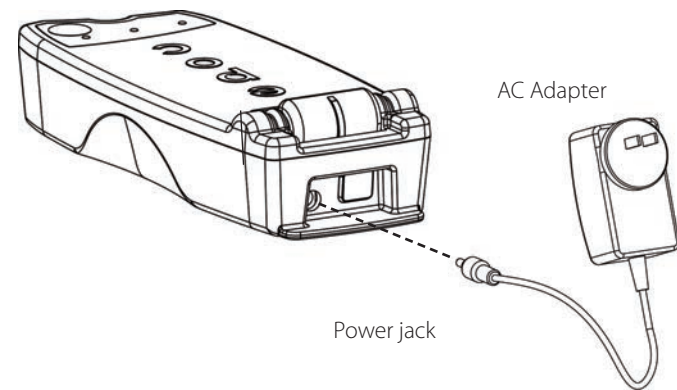

Step 2: Label

Label the epoc Reader to help easily identify it from other Readers in your facility. Use the included labels, and place the clear overlay over the label to increase its durability.

Note: Before placing this unit into service, consult your facility's internal quality control procedures and regulatory requirements.

© 2015 Alere. All rights reserved. The Alere Logo, Alere, and epoc are trademarks of the Alere group of companies. All other trademarks referenced are trademarks of their respective owners.